DE Diagnosewaage Gebrauchsanweisung	2	IT Bilancia diagnostica Istruzioni per l'uso.....	42
EN Diagnostic bathroom scale Instructions for use	12	TR Diyagnoz terazisi Kullanım kılavuzu	52
FR Pèse-personne impédancemètre Mode d'emploi.....	22	RU Диагностические весы Инструкция по применению.....	61
ES Báscula diagnóstica Instrucciones de uso.....	32	PL Waga diagnostyczna Instrukcja obsługi.....	70

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

With kind regards,
Your Beurer team

Contents

1. Included in delivery.....	13	8. Usage	17
2. Signs and symbols	13	9. Cleaning and maintenance.....	19
3. Intended use.....	14	10. Incorrect measurement	19
4. Warnings and safety notes	14	11. Disposal.....	20
5. Device description.....	16	12. Technical specifications	20
6. Initial use	16	13. Warranty/service.....	20
7. Entering user data	16		

Getting to know your device

Functions of the device

This digital scale is intended for weighing and providing a diagnosis of your personal fitness data. It is intended for private use.

The scale is equipped with the following functions, which can be used by up to 10 different people:

- Body weight measurement
- Body fat percentage calculation
- Body water content
- Muscle percentage.

The scale also includes the following additional functions:

- Switch between kilograms “kg”, pounds “lbs” and stones “st”
- Automatic switch-off function
- Low battery indicator.

The measurement principle

This scale works on the principle of B.I.A. (bioelectrical impedance analysis). This involves the calculation of body content in a matter of seconds, using a current that cannot be felt, is completely harmless and does not pose any risk.

When this measurement of the electrical resistance (impedance) is considered alongside constants and/or individual values (age, height, gender, activity level), the body fat percentage and other variables in the body can be calculated. Muscle tissue and water conduct electricity well and therefore have a lower resistance. In contrast, bones and fat tissue have low conductivity as the fat cells and bones are barely able to conduct the current due to very high levels of resistance.

Please note that the values calculated by the diagnostic bathroom scale only represent an approximation of the actual medical analytical values for the body. Only a specialist doctor can precisely establish the body fat, body water, muscle percentage and bone structure using medical methods (e.g. computer tomography).



1. Included in delivery


Check that the device packaging has not been tampered with and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

- BF 220 diagnostic bathroom scale
- 2 x 3V CR2032
- These instructions for use

2. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

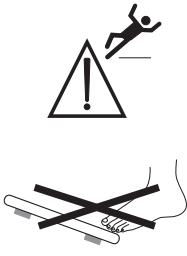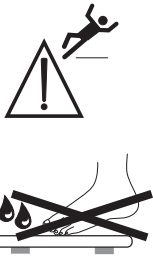	WARNING Warning notice indicating a risk of injury or damage to health.		
	IMPORTANT Safety note indicating possible damage to the device/accessory.		
	Note Note on important information		
	Observe the instructions for use		
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE		
	Do not dispose of batteries containing hazardous substances with household waste		
	Dispose of packaging in an environmentally friendly manner		
	This product satisfies the requirements of the applicable European and national directives.		
	The device may not be used by people with medical implants (e.g. pacemakers), as this may affect their functionality.		
	Manufacturer		
	Danger of tipping: position yourself in the centre of the weighing surface.		Danger of slipping: do not step on the scale with wet feet.

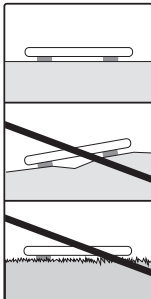	<p>Place the scale on an even surface. No carpet.</p>		<p>Do not load the scale beyond 180 kg / 396 lb / 28 st.</p>
--	---	--	--

3. Intended use

The device is only intended for weighing humans and for recording your personal fitness data. The device is only intended for private use, and not for medical or commercial purposes.

4. Warnings and safety notes

WARNING

- **The scale may not be used by people with medical implants (e.g. pacemakers) as this may affect their functionality.**
- Do not use during pregnancy.
- Do not step onto the outer edge of the scale on just one side: danger of tipping!
- Swallowing batteries can be extremely dangerous. Keep the batteries and scale out of the reach of small children. Should a battery be swallowed, seek medical assistance immediately.
- Keep packaging material away from children (risk of suffocation).
- Important: do not step onto the scale with wet feet or if the surface of the scale is damp – danger of slipping!

Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- **⚠ Choking hazard!** Small children may swallow and choke on batteries. Therefore, store batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.
- **⚠ Risk of explosion!** Do not throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries!
- Do not disassemble, open or crush the batteries.

General notes

- Please note that measuring tolerances are possible for technical reasons, as this scale is not calibrated for use in a professional medical context.
- Load capacity: max. 180 kg (396 lb, 28 st). Result in steps of 100g (0.2 lb, 1/4 st). Measurements of body fat, body water and muscle density in steps of 0.1%.
- When supplied to the customer, the scale is set to weigh and measure in “kg” and “cm”.

- Place the scale on an even, hard surface; a hard surface is crucial for achieving accurate measurements.
- Repairs may only be carried out by Beurer Customer Services or authorised retailers. Before submitting a complaint, please check the batteries first and replace them if necessary.

Storage and maintenance

The accuracy of the measurements and service life of the device depend on its careful handling:

IMPORTANT

- The device should be cleaned from time to time. Do not use any abrasive cleaning products and never submerge the device in water.
- Make sure that no liquids come into contact with the scale. Never submerge the scale in water. Never rinse it in running water.
- Do not place any objects on the scale when it is not in use.
- Do not press the button violently or with pointed objects.
- Do not expose the scale to high temperatures or strong electromagnetic fields.
- Protect the device from knocks, damp, dust, chemicals, marked temperature fluctuations and nearby sources of heat (ovens, heaters).

General tips

- As far as possible, weigh yourself at the same time of day (ideally in the morning), after having been to the toilet, on an empty stomach and without clothing, in order to obtain results which can be compared.
- Important point regarding the measurement: the calculation of body fat may only be made when bare-foot; the soles of the feet may be lightly dampened where appropriate. Results may be unsatisfactory if the soles of the feet are completely dry or have a large amount of hard skin since conductivity will be impaired.
- Stand up straight and still during the measurement.
- Wait a few hours after any physical exertion to which your body is unaccustomed.
- Wait approx. 15 minutes after getting up so that the water stored in the body can disperse.
- Remember that only the long-term trend is important. Short-term changes in weight within a few days are normally caused by loss of fluids; however, body water plays an important role in terms of well-being.

Limitations


When calculating body fat and the other values, unusual and implausible results may occur in the case of:

- Children under approx. 10 years of age
- Performance athletes and body-builders
- Persons with a fever, persons undergoing dialysis, persons with oedema symptoms and persons suffering from osteoporosis
- Persons taking cardiovascular medicine (affecting the heart and vascular system)
- Persons taking vasodilative or vasoconstrictive medication
- Persons with significant anatomical abnormalities in their legs compared with their overall body height (leg length significantly shorter or longer).

5. Device description

Overview

1. Electrodes
2. Down button ▼ (Pushbutton)
3. SET button (Pushbutton)
4. Up button ▲ (Pushbutton)
5. Display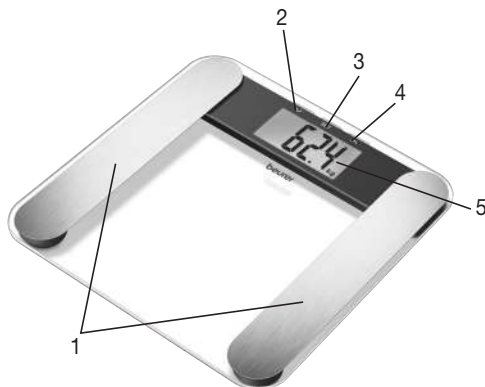

6. Initial use

Inserting the batteries


Remove the batteries from the protective packaging and insert the batteries into the scale (rear of the scale) ensuring that the battery polarity is correct. If the scale does not function, remove the batteries completely and reinsert them.

Changing the weight unit

When supplied to the customer, the scale is set to weigh in “kg”. Switch on the scale by briefly applying pressure to the standing surface with your foot (approx. 1-2 seconds). Wait until “0.0 kg” appears in the display. To the rear of the scale, there is a button where you can select “pound” and “stones” (lb, st).

Positioning the scale

Place the scale on an even, hard surface; a hard surface is crucial for achieving accurate measurements.

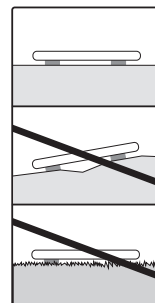7. Entering user data

To calculate your body fat percentage and additional body values, you must save the personal user data.

The scale has 10 user memory spaces which allow you and members of your family to save and access personal settings, for example.

- **Place the scale on an even hard surface; a hard surface is crucial for achieving accurate measurements.**
- Switch on the scale by briefly applying pressure to the standing surface with your foot (approx. 1-2 seconds). Wait until “0.0 kg” appears in the display.
- Start configuring the settings by pressing the SET button. The first memory space flashes in the display.
- Press the ▲/▼ button to select the desired memory space and confirm by pressing the SET button.
- You can now implement the following settings:

User data	Set values
Users/persons	P-1 to P-10
Height	100 to 220 cm
Age	10 to 100 years
Gender	Male (♂), female (♀)
Activity level	1 to 5

Activity levels

The medium-term and long-term perspective is decisive when selecting the activity level.

Activity level	Physical activity
1	None.
2	Low: little and light physical exertion (e.g. going for walks, light gardening, simple gymnastics).
3	Medium: physical exertion, at least 2 to 4 times a week for 30 minutes each.
4	High: physical exertion, at least 4 to 6 times a week for 30 minutes each.
5	Very high: intensive physical exertion, intensive training or hard physical work for at least 1 hour every day.

- Changing the values: Press either the  or the  button, or press and hold to quickly scroll through the options.
- Confirming the entries: Press the SET button.
- Once the values have been set, "0.0 kg" appears in the display.
- Now stand on the scale with bare feet whilst "0.0 kg" is displayed.
- Then the scale is ready for measuring. If you do not carry out a measurement, the scale switches off automatically after several seconds.

8. Usage

8.1 Taking measurements

Place the scale on an even, hard surface; a hard surface is crucial for achieving accurate measurements.

Measuring weight, carrying out diagnostics

Step onto the scale with bare feet and ensure that you are standing still with equal weight distribution and with both feet on the electrodes.

-  **Note:** Your feet, legs, calves and thighs must not touch each other. Otherwise the measurement cannot be performed correctly. The measurement result will be incorrect if the measurement is taken with socks on.

The scale begins to take measurements straight away.

The following data are displayed:

- Body weight in kg
- Body fat percentage BF in %
- Water percentage  in %
- Muscle percentage  in %
- Now all measured values are displayed consecutively and the scale switches off.

Only measuring weight

Step onto the scale wearing shoes. Stand still with your weight distributed equally on both feet. The scale begins to take measurements straight away.

Switching off the scale

The scale switches itself off automatically.

8.2 Evaluating the results

Body fat percentage

The following body fat percentages serve as a guideline (for further information please consult your doctor).

Male

Age	Low	Normal	Moderate	High
10–14	<11%	11–16%	16.1–21%	>21.1%
15–19	<12%	12–17%	17.1–22%	>22.1%
20–29	<13%	13–18%	18.1–23%	>23.1%
30–39	<14%	14–19%	19.1–24%	>24.1%
40–49	<15%	15–20%	20.1–25%	>25.1%
50–59	<16%	16–21%	21.1–26%	>26.1%
60–69	<17%	17–22%	22.1–27%	>27.1%
70–100	<18%	18–23%	23.1–28%	>28.1%

Female

Age	Low	Normal	Moderate	High
10–14	<16%	16–21%	21.1–26%	>26.1%
15–19	<17%	17–22%	22.1–27%	>27.1%
20–29	<18%	18–23%	23.1–28%	>28.1%
30–39	<19%	19–24%	24.1–29%	>29.1%
40–49	<20%	20–25%	25.1–30%	>30.1%
50–59	<21%	21–26%	26.1–31%	>31.1%
60–69	<22%	22–27%	27.1–32%	>32.1%
70–100	<23%	23–28%	28.1–33%	>33.1%

Values will often be lower for fitness enthusiasts. Depending on the sporting activities performed, the training intensity and the physical constitution, the results may still be below the stated standard values. Please note, however, that there may be a risk to health in the case of extremely low values.

Body water content

The body water content in % is normally within the following ranges:

Male

Age	Poor	Good	Very good
10–100	<50%	50–65%	>65%

Female

Age	Poor	Good	Very good
10–100	<45%	45–60%	>60%

Body fat contains relatively little water, therefore body water content may be below the standard values in persons with a high body fat percentage. On the other hand, endurance athletes may exceed the standard values due to having low levels of fat and a high muscle percentage.

The body water calculation performed using this scale is not suitable for drawing medical conclusions, for example in the case of age-related water retention. Consult your doctor where necessary. A high body water content is generally desirable.

Muscle percentage

The muscle percentage is normally within the following ranges:

Male

Age	Low	Normal	High
10–14	<44%	44–57%	>57%
15–19	<43%	43–56%	>56%
20–29	<42%	42–54%	>54%
30–39	<41%	41–52%	>52%
40–49	<40%	40–50%	>50%
50–59	<39%	39–48%	>48%
60–69	<38%	38–47%	>47%
70–100	<37%	37–46%	>46%

Female

Age	Low	Normal	High
10–14	<36%	36–43%	>43%
15–19	<35%	35–41%	>41%
20–29	<34%	34–39%	>39%
30–39	<33%	33–38%	>38%
40–49	<31%	31–36%	>36%
50–59	<29%	29–34%	>34%
60–69	<28%	28–33%	>33%
70–100	<27%	27–32%	>32%

Temporal context of the results

 **Note:** Please note that only the long-term trend is important. Brief deviations in weight within a few days are normally caused by loss of fluids.

The interpretation of the results is based on changes in total body weight, percentage of body fat, body water and muscle content, as well as on the length of time over which these changes occur.

Rapid changes within the scope of a few days are to be considered separately from medium-term changes (in the scope of weeks) and long-term changes (months).

It can be said as a basic rule that short-term changes in weight are almost entirely changes in water content, whereas medium-term and long-term changes may also relate to the fat percentage and the muscle percentage.

- If your weight drops in the short term but your body fat percentage increases or stays the same, you have only lost water, for example following a training session, visit to the sauna or a diet aimed only at fast weight loss.
- If your weight increases in the medium term and your body fat percentage drops or stays the same, you may have built up valuable muscle mass.
- If your weight and body fat percentage both fall at the same time, then your diet is working – you are losing fat mass.
- Ideally, you should support your diet with physical activity, fitness or strength training. This enables you to increase your muscle percentage in the medium term.
- Body fat, body water and muscle percentages should not be totalled (muscle tissue also contains components made of body water).

8.3 Other functions

Replacing the batteries

Your scale is equipped with a low battery indicator. If you operate the scale with flat batteries, “Lo” will appear on the display and the scale will automatically switch off. In this case, the batteries must be replaced (2x 3V CR 2032).

 **NOTE:**

- When replacing the batteries, use batteries of the same type, make and capacity.
- Do not use rechargeable batteries.
- Use batteries free from heavy metals.

9. Cleaning and maintenance

The device should be cleaned from time to time.

This should be done using a damp cloth and, if necessary, a small amount of detergent.

 **IMPORTANT**

- Never use abrasive solvents or cleaning products!
- Never submerge the device in water!
- Do not clean the device in a dishwasher!

10. Incorrect measurement

If the scale detects an error during weighing, “FFFF” or “Err” appears in the display.

If you step onto the scale before “0.0” appears in the display, the scale will not operate properly.

Possible causes of errors:	Remedy:
– The maximum load-bearing capacity of 180 kg was exceeded.	– Only weigh the maximum permissible weight.
– The electrical resistance between the electrodes and the soles of your feet is too high (e.g. with heavily callused skin).	– Repeat weighing barefoot. – Slightly moisten the soles of your feet if necessary. Remove the calluses on the soles of your feet if necessary.
– Your body fat lies outside the measurable range (less than 5% or greater than 55%).	– Repeat weighing barefoot. – Slightly moisten the soles of your feet if necessary.

11. Disposal

Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

The codes below are printed on batteries containing harmful substances:

Pb = Battery contains lead

Cd = Battery contains cadmium

Hg = Battery contains mercury

For environmental reasons, do not dispose of the device in the household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country.

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

Dispose of packaging in an environmentally friendly manner.

12. Technical specifications

Model:	BF 220
Dimensions:	32 x 32 x 2,4 cm
Weight:	2300 g
Measurement range:	5-180 kg
Scale interval d:	The display can be read in increments of 0.1 kg.
Repetition accuracy:	The measuring tolerance for repeated measurements is +/- 0.4 kg (several measurements in direct succession on the same scale with position of scale and person the same as far as possible).
Absolute precision:	In comparison with a calibrated weight, the measured value is +/- 1% +0.1 kg. E.g. at 40 kg this corresponds to +/- 0.5 kg; at 100 kg this corresponds to +/- 1.1 kg.

Subject to technical changes.

13. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as "Beurer") provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller's statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 5 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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